sigvaris

Compreflex Standard

INELASTIC WRAP - ARM

INSTRUCTIONS FOR USE AND CARE

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
Compreflex Standard Arm Sizing Chart (A–C) Tableau de Taillage (cm)

	Small	Medium	Large	X Large
C	23–32cm	28–37cm	33–43cm	39–49cm
B1	22–30cm	26–34cm	30–39cm	35–44cm
B	20–27cm	24–31cm	28–35cm	32–39cm
A	14–19cm	15–20cm	16–21cm	18–23cm

Valid for Rest of the World

Product Measurement and Size Chart:

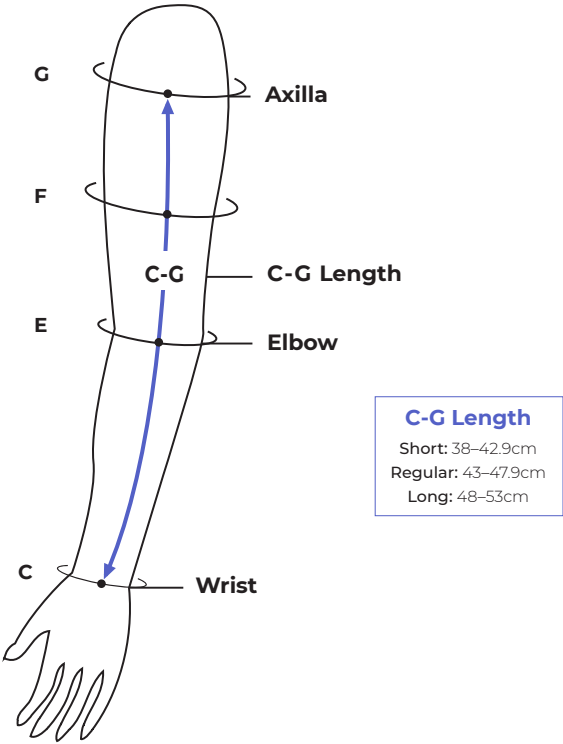Identifying the optimal product size ensures a proper fit and the correct distribution of compression. Measurements should be taken by a healthcare professional.

C-G: Measure length along dorsal aspect

Compreflex Standard Arm Sizing Chart (C-G)				
	Small	Medium	Large	X Large
G	23–32cm	28–37cm	33–43cm	39–49cm
F	22–30cm	26–34cm	30–39cm	35–44cm
E	20–27cm	24–31cm	28–35cm	32–39cm
C	14–19cm	15–20cm	16–21cm	18–23cm

Compreflex Standard:

Inelastic Wrap - Arm

en	Medical Device
fr	<i>Dispositif médical</i>
es	<i>Producto sanitario</i>
it	<i>Dispositivo medico</i>
de	<i>Medizinprodukt</i>
pt	<i>Dispositivo médico</i>
pl	<i>Wyrób medyczny</i>
nl	<i>Medisch hulpmiddel</i>
da	<i>Medicinsk udstyr</i>
fi	<i>Lääkinnällinen laite</i>
ru	<i>Медицинское изделие</i>
lt	<i>Medicinos priemonė</i>
sl	<i>Medicinski pripomoček</i>
tr	<i>Tıbbi cihaz</i>
ja	医療用製品
ko	의료 기기
ar	الجهاز الطبي

Compreflex® Standard: Inelastic Wrap - Arm

Product Descriptor:

The Compreflex Standard Arm is an inelastic compression device, offering variable adjustable compression of the arm. The device features opposing straps for the application of gradient compression.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic edema. The product is intended for multiple use.

Indications:

Lymphedema, Edema, Lipedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia cerulea dolens
- Severe PAOD of the arm, as confirmed by bilateral blood pressure measurement
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- Mild PAOD of the arm, as confirmed by bilateral blood pressure measurement
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic edema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or edema, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

1 Compreflex Standard Arm

1 Pair of liners

1 Butterfly strap

Material Composition:

- 39% Polyurethane foam
- 32% Polyamide
- 20% Cotton
- 5% Neoprene
- 4% Elastane

Care:

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Standard Arm device should be washed as often as needed

Caring for Your Liners:

- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Remove all jewelry from hand, wrist, and arm. With palm facing up, slide your arm into the provided liner
2. Gently slide the Compreflex Standard Arm onto arm, aligning the elbow crease with the elbow seams
3. Beginning at wrist, fasten all straps with a low tension. Readjust all the straps to a comfortable compression level
4. Attach Butterfly Strap to elbow crease area
5. To remove garment, start at the wrist and roll the straps back on themselves. Once all straps are secured, slide garment off the arm and remove the liner

Use:

While the Compreflex Standard Arm is suitable for day and nighttime wear, the garment should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the prescribing provider's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

Compreflex Standard: Inelastic Wrap - Arm

Product Descriptor:

The Compreflex Standard Arm is an inelastic compression device, offering variable adjustable compression in the arm. The device features opposing straps for the application of gradient compression.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic oedema. The product is intended for multiple use.

Indications:

Lymphoedema, Oedema, Lipoedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia coerulea dolens
- Severe PAOD of the arm, as confirmed by bilateral blood pressure measurement
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- Mild PAOD of the arm, as confirmed by bilateral blood pressure measurement
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic oedema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or oedema, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

1 Compreflex Standard Arm

1 Pair of liners

1 Butterfly strap

Material Composition:

- 39% Polyurethane foam
- 32% Polyamide
- 20% Cotton
- 5% Neoprene
- 4% Elastane

Care:

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Standard Arm device should be washed as often as needed

Caring for Your Liners:

- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Remove all jewelry from hand, wrist, and arm. With palm facing up, slide your arm into the provided liner
2. Gently slide the Compreflex Standard Arm onto arm, aligning the elbow crease with the elbow seams
3. Beginning at wrist, fasten all straps with a low tension. Readjust all the straps to a comfortable compression level
4. Attach Butterfly Strap to elbow crease area
5. To remove garment, start at the wrist and roll the straps back on themselves. Once all straps are secured, slide garment off the arm and remove the liner

Use:

While the Compreflex Standard Arm is suitable for day and night time wear, the garment should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the prescribing provider's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

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St. Gallen/Switzerland, in many countries worldwide.    2021 Copyright SIGVARIS, Inc.
   Sigvaris and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014,
SIGVARIS AG
SIGVARIS GmbH