sigvaris

Compreflex Standard

INELASTIC WRAP - KNEE

INSTRUCTIONS FOR USE AND CARE

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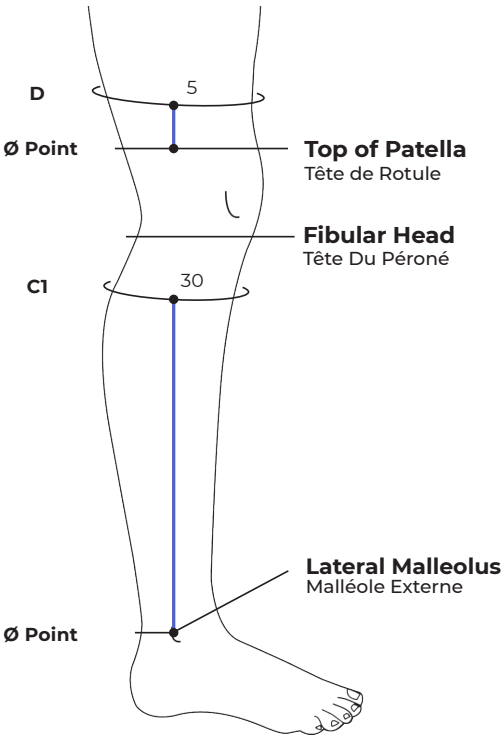
Compreflex Standard Knee Sizing Chart (C1-D)


Tableau De Taillage (cm)

	Small	Medium	Large	X Large
D	38–48cm	46–56cm	53–63cm	58–73cm
C1	29–39cm	34–44cm	39–49cm	44–65cm

Valid for Rest of the World

Product Measurement and Size Chart:

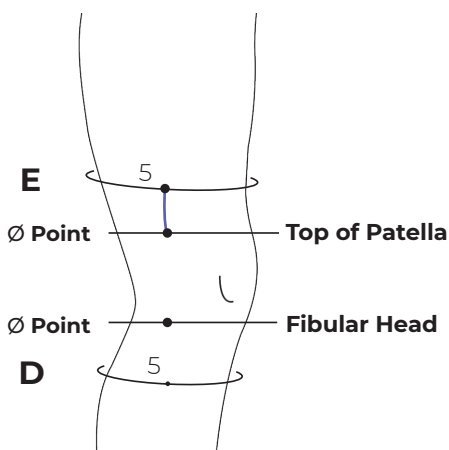Identifying the optimal product size ensures a proper fit and the correct distribution of compression. Measurements should be taken by a healthcare professional.

Compreflex Standard Knee Sizing Chart (D-E)

	Small	Medium	Large	X Large
E	38–48cm	46–56cm	53–63cm	58–73cm
D	29–39cm	34–44cm	39–49cm	44–65cm

Compreflex Standard:

Inelastic Wrap - Knee

en	Medical Device
fr	<i>Dispositif médical</i>
es	<i>Producto sanitario</i>
it	<i>Dispositivo medico</i>
de	<i>Medizinprodukt</i>
pt	<i>Dispositivo médico</i>
pl	<i>Wyrób medyczny</i>
nl	<i>Medisch hulpmiddel</i>
da	<i>Medicinsk udstyr</i>
fi	<i>Lääkinnällinen laite</i>
ru	<i>Медицинское изделие</i>
lt	<i>Medicinos priemonė</i>
sl	<i>Medicinski pripomoček</i>
tr	<i>Tıbbi cihaz</i>
ja	医療用製品
ko	의료 기기
ar	الجهاز الطبي

Compreflex® Standard: Inelastic Wrap - Knee

Product Descriptor:

The Compreflex Standard Knee is an inelastic compression device, providing accurate compression levels of 20–50mmHg to the knee, using the measurement device Accutab. The Accutab is also aligned to the RAL compression ranges of 21–46mmHg. The device includes a Butterfly Strap, used to compress the patella and other regions where edema may pocket.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic edema. The product is intended for multiple use.

Indications:

Lymphedema, Edema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia cerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic edema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or edema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

- 1 Compreflex Standard Knee
- 1 Set of Accutab® Devices
- 1 Butterfly Strap

Material Composition:

- 65% Polyurethane Foam
- 32% Polyamide
- 3% Elastane

Care:

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Standard Knee device should be washed as often as needed

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Open and roll back each strap so it attaches to the garment
2. Next, slide garment on and pull over knee. The stretch panel will secure garment in place
3. Starting with the bottom strap, gently lay the straps closed over the knee without applying compression, making sure there is minimal strap overlap and no gaps between straps
4. Press the velcro ends of the strap firmly onto the sides of the garment. Continue securing the remaining straps in the same manner
5. Apply the Accutab Devices and finish tightening the straps according to the instructions on the provided Accutab Devices insert
6. If desired, place the Butterfly Strap (included) over the patella and attach to garment to cover any gaps between straps
7. To remove garment, detach the Butterfly Strap. Open and roll back each strap so it attaches to the garment and slide garment off the leg

Use:

While the Compreflex Standard Knee is suitable for day and nighttime wear, it should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the medical professional's recommendations. SIGVARIS GROUP recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

Compreflex Standard: Inelastic Wrap - Knee

Product Descriptor:

The Compreflex Standard Knee is an inelastic compression device, providing accurate compression levels of 20–50mmHg to the knee. The Accutab is also aligned to the RAL compression ranges of 21–46mmHg. The device includes a Butterfly Strap, used to compress the patella and other regions where edema may pocket.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic oedema. The product is intended for multiple use.

Indications:

Lymphoedema, Oedema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipoedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia coerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic oedema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or oedema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

- 1 Compreflex Standard Knee
- 1 Set of Accutab® Devices
- 1 Butterfly Strap

Material Composition:

- 65% Polyurethane Foam
- 32% Polyamide
- 3% Elastane

Care:

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Standard Knee device should be washed as often as needed

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Open and roll back each strap so it attaches to the garment
2. Next, slide garment on and pull over knee. The stretch panel will secure garment in place.
3. Starting with the bottom strap, gently lay the straps closed over the knee without applying compression, making sure there is minimal strap overlap and no gaps between straps
4. Press the velcro ends of the strap firmly onto the sides of the garment. Continue securing the remaining straps in the same manner
5. Apply the Accutab Devices and finish tightening the straps according to the instructions on the provided Accutab Devices insert
6. If desired, place the Butterfly Strap (included) over the patella and attach to garment to cover any gaps between straps
7. To remove garment, detach the Butterfly Strap. Open and roll back each strap so it attaches to the garment and slide garment off the leg

Use:

While the Compreflex Standard Knee is suitable for day and night time wear, it should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the medical professional's recommendations. SIGVARIS GROUP recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

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® Sigvaris and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014,
St. Gallen/Switzerland, in many countries worldwide. © 2021 Copyright SIGVARIS, Inc.
SIGVARIS AG
SIGVARIS GmbH